CLINICAL CHEMISTRY STANDARDS

PRESENTACION / CONTENTS	PATRONES DE QUIMICA CLINICA CLINICAL CHEMISTRY STANDARDS		
<table border="1" style="width: 100%; border-collapse: collapse;"> <tr> <td style="text-align: center; padding: 5px;">REF</td> <td style="padding: 5px;">Ver recuadro inferior See description below</td> </tr> </table>		REF	Ver recuadro inferior See description below
REF		Ver recuadro inferior See description below	
Sólo para uso diagnóstico <i>in vitro</i> Only for <i>in vitro</i> diagnostic use			

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INTENDED USE

These reagents are intended for the calibration of the quantitative determinations of clinical chemistry parameters.
The use of a standard to calculate results allow obtaining an accuracy independent of the system or instrument used.

COMPOSITION

CAL Bilirubin Standard: Freeze-dried protein-based calibrator.
Hemoglobin Standard: Liquid animal-based calibrator.
Other standards: Liquid organic matrix based primary standards.

STABILITY

The standards are stable until the expiration date printed on the label when stored tightly closed and contamination during their use is avoided.

Storage and stability stated on the label.

Signs of deterioration.

Presence of particles, turbidity and/or contamination.

PREPARATION

Liquid standards: Ready to use.
Freeze-dried standards: Reconstitution stated on the vial.

PROCEDURE

Follow the technical instructions of the corresponding kit.

NOTES

- Standards may cause a systematic error with some automated procedures when pumping devices are used. The value or set point suggested by the manufacturer may not necessary be correct for the equipment and method used.
- Use clean disposable pipette tips for its dispensation.

REF	DESCRIPTION
1933105	LDL-Cholesterol Standard 50 mg/dL 1x3 mL
1944005	Mg 2 mg/dL-Ca 10 mg/dL
1943005	Lipase Calibrator 1x3 mL
1962005	Protein (Urine and CSF) Standard 2 g/L 1x3 mL
1953005	Protein Total Standard 7 g/dL 1x3 mL
1955005	Triglycerides Standard 200 mg/dL 1x3 mL
1958005	Urea Standard 50 mg/dL 1x3 mL
1961005	Uric acid Standard 6 mg/dL 1x3 mL

B19-5/1512
R1 ing cast