STEL 3-Cleaner

CONTENTS / CONTENIDO	
REF	3802015 STEL 3-Cleaner 1 x 1 L
Only for professional <i>in vitro</i> diagnostic use Sólo para uso profesional de diagnóstico <i>in vitro</i>	

STEL 3-Cleaner

STEL 3-Solución de limpieza

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PRINCIPLE

STEL 3-Cleaner is used on STEL 3 Auto hematology analyzer for cleaning and washing tubing system, removing blood residual and other particles to ensure blood cell counting.

REAGENT COMPOSITION

Buffer solution <0.3%, Protease <0.2%.

STORAGE AND STABILITY

⚠ Store at 5-35°C. Relative humidity <90%. Store the vials tightly closed, protected from light and prevented contaminations during the use. Do not put it in refrigerator.

After opening, the reagent is valid for 60 days.

Discard if appear signs of deterioration:

- Presence of particles and turbidity.

INSTRUCTIONS FOR USE

1. Open the external packaging material, connect pipeline of analyzer with mouth of reagent bottle.
2. When using the product, please refer to instructions for the STEL 3 auto hematology analyzer.

NOTES

1. Dispose the waste and contaminated packing material according to local laws and regulations.
2. Test results may be affected under following conditions:
 - The reagent is decayed or expired;
 - The reagent is contaminated by dust, etc.
 - The reagent is used together with reagent manufactured by other companies.
3. This reagent is only for in-vitro diagnosis, do not drink it, please put on gloves during operation. In case of contact with skin, eyes and mouth, rinse with plenty of water immediately. See the doctor if necessary.

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