STEL 3- Extra Cleaner

CONTENTS / CONTENIDO	
REF	3802020 STEL 3- Extra Cleaner 1 x 0.1 L
Only for professional <i>in vitro</i> diagnostic use Sólo para uso profesional de diagnóstico <i>in vitro</i>	

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PRINCIPLE

STEL 3- Extra Cleaner is used on STEL 3 Auto hematology analyzer for daily maintenance and system cleaning.

REAGENT COMPOSITION

5.0% sodium hypochlorite

STORAGE AND STABILITY

⚠ Store at 5-35°C. Relative humidity <90%. Store the vials tightly closed, protected from light and prevented contaminations during the use. Do not put it in refrigerator.

After opening, the reagent is valid for 60 days.

Discard if appear signs of deterioration:

- Presence of particles, precipitates, floccules or turbidity.

INSTRUCTIONS FOR USE

When the system give the message to add concentrate cleanser, add it manually into de chamber. This operation shall be perform once every three days. See STEL-3 User manual for more information.

NOTES

1. Dispose the waste and contaminated packing material according to local laws and regulations.
2. Test results may be affected under following conditions:
 - The reagent is decayed or expired;
 - The reagent is contaminated by dust, etc.
 - The reagent is used together with reagent manufactured by other companies.
3. This reagent is only for in-vitro diagnosis, do not drink it, please put on gloves during operation. In case of contact with skin, eyes and mouth, rinse with plenty of water immediately. See the doctor if necessary.

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