# us-CRP Turbidimetric Control

<b>CONTENTS / CONTENIDO</b>	
REF	3925010 us-CRP Turbidimetric Control 1 x 2 mL
Only for <i>in vitro</i> diagnostic use Sólo para uso diagnóstico <i>in vitro</i>	

## us-CRP Turbidimetric Control Control de us-PCR Turbidimétrico

### *Turbidimetry / Turbidimetria*

#### PRINCIPLE

us-CRP Turbidimetric Control is a solution of human CRP suitable for the accuracy and precision control of determinations of low CRP concentration level by turbidimetry in clinical laboratories.

#### REAGENT COMPOSITION

##### us-CRP Turbidimetric Control.

Human CRP. Sodium azide 0.95 g/L.

**Precautions.** The reagent, from human origin, has been given negative results to anti-HIV 1/2, HBsAg and anti-HCV. Handle cautiously is recommended.

The reagent contains sodium azide. Avoid any contact with skin or mucous.

#### REAGENT PREPARATION

us-CRP Turbidimetric Control. Ready- to- use.

#### STORAGE AND STABILITY

- Store at 2-8°C.  
The reagent is stable until the expiration date printed on the label. Keep the control tightly closed at 2-8°C and prevent contaminations during its use.  
Do not use the control after the expiration date.
- Presence of particles and turbidity is sign of deterioration.
- Opened vials are stable for 30 days at 2-8°C.

#### INSTRUCTIONS FOR USE

Use the Control as a patient sample following the instructions of the corresponding analytical method.

#### ASSIGNED VALUES

The assigned concentration value and its interval, indicate on the label, has been assigned by turbidimetry method with an appropriate instrument, and it is given for orientation only. Results may slightly differ from these values because of the different methods and instruments used in the different laboratories. It is useful that every laboratory establishes its own precision parameters.

#### NOTES

- Bacterial contamination may cause reductions in the stability of many components of the Control.
- When pipetting the Control to the instrument avoid the formation of foam, a common cause of erroneous results.

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