# D-Dimer Turbidimetric Control

<b>CONTENTS / CONTENIDO</b>	
<b>REF</b>	3945010 D-Dimer Turbidimetric Control 2 x 0.5 mL
Only for <i>in vitro</i> diagnostic use Sólo para uso diagnóstico <i>in vitro</i>	

## D-Dimer Turbidimetric Control Control de D-Dimer Turbidimétrico

*Turbidimetry / Turbidimetría*

### PRINCIPLE

D-Dimer Turbidimetric Control set is a lyophilized solution of D-Dimer suitable for the accuracy and precision control in the determination of D-Dimer by turbidimetric method.

### REAGENT COMPOSITION

- L1** Ref. 3945011. Level 1 1 x 0.5 mL
- L2** Ref. 3945012. Level 2 1 x 0.5 mL

Lyophilized solution of highly purified human D-Dimer. Contains bovine albumin, stabilizers and preservative.

**Precautions.** The reagent, from human origin, has been given negative results to anti-HIV 1/2, HBsAg and anti-HCV. Handle cautiously is recommended. Contains sodium azide. Avoid any contact with skin or mucous.

### REAGENT PREPARATION

Lyophilized. Dissolve the contents with exactly **0.5 mL** of distilled water. Replace the stopper and swirl carefully. Let stand the vial at 15-25°C for 20-30 minutes before use. Do not shake.

### STORAGE AND STABILITY

1. Store at 2-8°C. Stable until the expiration date printed on the label. Keep the control tightly closed at 2-8°C and prevent contaminations during its use.  
Do not use the control after the expiration date.
2. Stability after reconstitution: 2 days at 15-25°C, 15 days at 2-8°C and 2 months at -20°C.
3. Presence of particles and turbidity is sign of deterioration.

### INSTRUCTIONS FOR USE

Use the Control as a patient sample following the instructions of the corresponding analytical method.

### NOTES

1. Bacterial contamination may cause reductions in the stability of many components of the Control.
2. When pipetting the Control to the instrument avoid the formation of foam, as it may cause of erroneous results.

### ASSIGNED VALUES

The concentration value and range have been assigned using D-Dimer Turbidimetric Linear reagents. However, the results may slightly differ from these values because of the different instruments used in the different laboratories. It is useful that every laboratory establishes its own precision parameters.

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