

MET Methamphetamine strip

CONTENTS		
REF	4435150 Methamphetamine	50 tests
For professional <i>in vitro</i> diagnostic use only		

Methamphetamine

A rapid test for the qualitative detection of Methamphetamine in human urine.

ONE STEP

PRINCIPLE

The LINEAR MET Methamphetamine strip is an immunoassay based on the principle of competitive binding. Drugs which may be present in the urine specimen compete against the drug conjugate for binding sites on the antibody.

During testing, a urine specimen migrates upward by capillary action. Methamphetamine, if present in the urine specimen below 1,000 ng/mL, will not saturate the binding sites of antibody coated particles in the test strip. The antibody-coated particles will then be captured by immobilized Methamphetamine conjugate and a visible colored line will show up in the test line region. The colored line will not form in the test line region if the Methamphetamine level exceeds 1,000 ng/mL because it will saturate all the binding sites of anti-Methamphetamine antibodies. A drug-positive urine specimen will not generate a colored line in the test line region, while a drug-negative urine specimen or a specimen containing a drug concentration less than the cut-off will generate a line in the test line region. To serve as a procedural control, a colored line will always appear at the control line region indicating that proper volume of specimen has been added and membrane wicking has occurred.


REAGENT COMPOSITION

MET Methamphetamine test strip, contains mouse monoclonal anti-Metamphetamine antibody-coupled particles and Metamphetamine - protein conjugate. A goat antibody is employed in the control line system.

PACKAGING CONTENTS

REF 4435150 50 MET Methamphetamine test strip.

STORAGE AND STABILITY

 Store at 2-30°C.

The test strips is stable through the expiration date printed on the sealed pouch. The test strip must remain in the sealed pouch until use. **DO NOT FREEZE.** Do not use beyond the expiration date.

SPECIMEN COLLECTION AND PREPARATION

The urine specimen must be collected in a clean and dry container. Urine collected at any time of the day may be used. Urine specimens exhibiting visible particles should be centrifuged, filtered, or allowed to settle to obtain clear specimen for testing.

Urine specimens may be stored at 2-8°C for up to 48 hours prior to testing. For long-term storage, specimens may be frozen and stored below -20°C. Frozen specimens should be thawed and mixed before testing.


MATERIAL REQUIRED

- Timer.
- Specimen collection container.

PROCEDURE

Allow the test strip, urine specimen, and/or controls to reach room temperature (15-30°C) prior to testing.

- Bring the pouch to room temperature before opening it. Remove the test strip from the sealed pouch and use it as soon as possible.
- With arrows pointing toward the urine specimen, **immerse the test strip vertically in the urine specimen for at least 10-15 seconds.** Do not pass the maximum line (MAX) on the test strip when immersing the strip. See the illustration below.
- Place the test strip on a non-absorbent flat surface, start the timer and wait for the red line(s) to appear. The result should be **read at 5 minutes.** Do not interpret the result after 15 minutes.

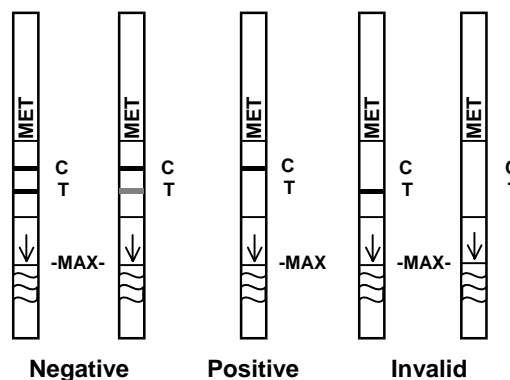

NEGATIVE: * **Two lines appear.** One colored line should be in the control region (C), and another apparent colored line should be in the test region (T). This negative result indicates that the Metamphetamine concentration is below the detectable level (1,000 ng/mL).

* **NOTE:** The shade of color in the test line region (T) may vary, but it should be considered negative whenever there is even a faint colored line.

POSITIVE: **One colored line appears in the control region (C).** No line appears in the test region (T). This positive result indicates that the Metamphetamine concentration exceeds the detectable level (1,000 ng/mL).

INVALID: **Control line fails to appear.** Insufficient specimen volume or incorrect procedural techniques are the most likely reasons for control line failure. Review the procedure and repeat the test using a new test strip. If the problem persists, discontinue using the lot immediately and contact your local distributor.

QUALITY CONTROL

A procedural control is included in the test. A red line appearing in the control region (C) is considered an internal procedural control. It confirms sufficient specimen volume, adequate membrane wicking and correct procedural technique. Control reagents are not supplied with this kit; however, it is recommended that positive and negative controls be tested as good laboratory testing practice to confirm the test procedure and to verify proper test performance.

CLINICAL SIGNIFICANCE

Methamphetamine is an addictive stimulant drug that strongly activates certain systems in the brain. Methamphetamine is closely related chemically to Amphetamine, but the central nervous system effects of Methamphetamine are greater. Methamphetamine is made in illegal laboratories and has a high potential for abuse and dependence. The drug can be taken orally, injected, or inhaled. Acute higher doses lead to enhanced stimulation of the central nervous system and induce euphoria, alertness, reduced appetite, and a sense of increased energy and power. Cardiovascular responses to Methamphetamine include increased blood pressure and cardiac arrhythmias. More acute responses produce anxiety, paranoia, hallucinations, psychotic behavior, and eventually, depression and exhaustion.

The effects of Methamphetamine generally last 2-4 hours, and the drug has a half-life of 9-24 hours in the body. Methamphetamine is excreted in the urine primarily as Amphetamine, and oxidized and deaminated derivatives. However, 10-20% of Methamphetamine is excreted unchanged. Thus, the presence of the parent compound in the urine indicates Methamphetamine use. Methamphetamine is generally detectable in the urine for 3-5 days, depending on urine pH level.

The LINEAR MET Methamphetamine strip is a rapid urine-screening test that can be performed without the use of an instrument. The test utilizes a monoclonal antibody to selectively detect elevated levels of Methamphetamine in urine. The LINEAR One Step Methamphetamine Test Strip (Urine) yields a positive result when the Methamphetamine in urine exceeds 1,000 ng/mL. Test to monitor therapeutic measures related to the study and control of detoxification treatments of drug of abuse and its effects in laboratory tests.

Dexbrompheniramine	Sodium Chloride
Dextromethorphan	Sulindac
4-Dimethylaminoantipyrine	Thioridazine
Dopamine	Trifluoperazine
Erythromycin	Trimethobenzamide
Ethanol	Tyramine
Furosemide	Vitamin C
Glucose	

ANALYTICAL PERFORMANCE

A. Accuracy

The accuracy of the LINEAR MET Methamphetamine strip was compared and checked against a commercially available test with a threshold value of 1,000 ng/mL. 120 urine samples taken from volunteer test persons who claimed to be non-consumers were examined with both tests. The results were 100% in agreement.

B. Reproducibility

The reproducibility of the LINEAR MET Methamphetamine strip was verified by blind tests performed at four different locations. All 60 utilized samples with a MET-concentration of 250 ng/mL yielded a negative result. All 60 samples with a MET-concentration of 2,000 ng/mL yielded a positive result. No significant differences were observed between test results of the different evaluation sites.

C. Precision

Test precision was determined by blind tests with control solutions. Controls with a MET-concentration of 250 ng/mL yield negative results. Controls with a MET-concentration of 1,500 ng/mL provide positive results.

D. Specificity

The specificity of the LINEAR MET Methamphetamine strip was tested with the substances listed below, all of which can be found in a normal urine specimen. These substances were added to normal drug free urine.

The following compounds with a similar chemical structure yielded a positive result at the specified concentration:

COMPOUND	CONCENTRATION (ng/mL)
D-Methamphetamine	1,000
D-Amphetamine	>40,000
Chloroquine	10,000
(+/-)-Ephedrine	>100,000
L-Methamphetamine	15,000
3,4-Methylenedioxy-amphetamine (MDA)	>10,000
3,4-Methylenedioxymethamphetamine (MDMA)	2,000
3,4-Methylenedioxyethyl-amphetamine (MDEA)	20,000
Procaine	100,000

All following listed compounds reacted negative up to a concentration of 100 µg/mL.

Acetamidophene	Guaiacol Glyceryl Ether
Acetone	Hemoglobin
Albumin	Imipramine
Amitriptyline	(+/-)-Isoproterenol
Ampicillin	Lidocaine
Aspartame	(+)-Naproxen
Aspirin	Oxalic Acid
Atropine	Penicillin-G
Benzocaine	Pheniramine
Bilirubin	Phenothiazine
Caffeine	Phenylethylamine
Chloroquine	Procaine
(+/-)-Chlorpheniramine	Quinidine
Chlorpheniramine	Ranitidine
Creatine	Riboflavine

NOTES

1. The LINEAR MET Methamphetamine strip provides only a qualitative, preliminary analytical result. A secondary analytical method must be used to obtain a confirmed result. Gas chromatography/mass spectrometry (GC/MS) is the preferred confirmatory method.^{1,2}
2. Adulterants, such as bleach and/or alum, in urine specimens may produce erroneous results regardless of the analytical method used. If adulteration is suspected, the test should be repeated with another urine specimen.
3. A positive result indicates presence of the drug or its metabolites but does not indicate level of intoxication, administration route or concentration in urine.
4. A negative result may not necessarily indicate drug-free urine. Negative results can be obtained when drug is present but below the cut-off level of the test.
5. Test does not distinguish between drugs of abuse and certain medications.

REFERENCES

1. Baselt, R.C. Disposition of Toxic Drugs and Chemicals in Man, Biomedical Publications, 1982
2. Urine Testing for Drugs of Abuse. National Institute on Drug Abuse (NIDA), Research Monograph 73, 1986
3. Thomas L. eds., Labor und Diagnose, 6. ed., TH-Books Verlagsgesellschaft, Frankfurt, 2005
4. Fed. Register, Department of Health and Human Services, Mandatory Guidelines for Federal Workplace Drug Testing Programs, 53, 69, 11970, 1988
5. McBay, A.J. Clin. Chem. 33, 33B-40B, 1987
6. Gilman, A.G., & Goodman, L.S. The Pharmacological Basis of Therapeutics, eds. MacMillan Publishing, New York, NY, 1980.
7. Minden, Sandra v.; Minden, Wolfgang v.; Analytik von Drogen und Medikamenten, von Minden GmbH, Moers 2002
8. Oyler, Jonathan M.; Cone, Edward J.; Joseph, Robert E.; Moolchau, Eric, T.; Huestis, Marylin A.: Duration of Detectable Metamphetamine and Amphetamine Excretion in Urine after Controlled Oral Administration of Methamphetamine to Humans, Clinical Chemistry 48:10, 1703-1714 (2002)

