

# FRUCTOSAMINE STANDARD

REF CT19460

2 x 3 mL

## CONTENTS

2 x 3 mL

For *in vitro* diagnostic use only

## APLICACIONES

Reactiv para la calibración de determinaciones cuantitativas de fructosamina.

La utilización de un patrón para calcular los resultados permite obtener una exactitud independiente del fotómetro y del sistema de lectura empleados.

## COMPOSICIÓN

**CAL** Patrón líquido primarios de base orgánica.

## ESTABILIDAD

Estable hasta la fecha indicada en la etiqueta si se mantienen bien cerrado y se evita su contaminación.

Conservación y estabilidad indicada en la etiqueta.

### Indicadores de deterioro:

Presencia de partículas, turbidez y contaminación.

## PREPARACION

Listo para su uso.

## TECNICA

Seguir las instrucciones de uso indicadas en los kits correspondientes.

## NOTAS

- Los patrones pueden causar un error sistemático con algunas técnicas automatizadas en las que intervienen jeringas dosificadoras.
- El valor central o punto de ajuste sugerido por el fabricante puede no ser necesariamente correcto para un instrumento y método determinado.
- Usar puntas de pipeta desechables limpias para su dispensación.

## INTENDED USE

Reagent for calibration of the quantitative determinations of fructosamine.

The use of a standard to calculate results allow obtaining an accuracy independent of the system or instrument used.

## COMPOSITION

**CAL** Liquid organic matrix based primary standard.

## STABILITY

The standards are stable until the expiration date printed on the label when stored tightly closed and contamination during their use is avoided.

Storage and stability stated on the label.

### Signs of deterioration:

Presence of particles, turbidity and contamination.

## PREPARATION

Ready to use.

## PROCEDURE

Follow the technical instructions of the corresponding kit.

## NOTES

- Standards may cause a systematic error with some automated procedures when pumping devices are used.
- The value or set point suggested by the manufacturer may not necessarily be correct for the equipment and method used.
- Use clean disposable pipette tips for its dispensation.

