GGT BR

IFCC


KINETIC

Quantitative determination of γ -GT activity.

REF KR10190 1 x 40 mL	REF KR10192 5 x 40 mL
CONTENTS R1. Reagent 1 x 32 mL R2. Reagent 1 x 8 mL	CONTENTS R1. Reagent 5 x 32 mL R2. Reagent 5 x 8 mL
For professional <i>in vitro</i> diagnostic use only	

SUMMARY

Gamma-glutamyltransferase (γ -GT) catalyzes the transfer of a γ -glutamyl group from γ -glutamyl-3-carboxy-4-nitroanilide to glycylglycine with the formation of L- γ -glutamyl-glycylglycine and 5-amino-2-nitro-benzoate. The amount of 5-amino-2-nitro-benzoate formed is proportional to the enzyme activity present in the sample monitored kinetically at 405 nm.¹

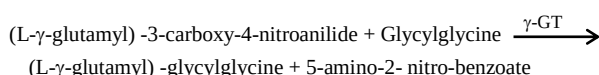REAGENTS

R1. TRIS 133 mmol/L, glycylglycine 138 mmol/L, sodium azide 0.09%.

R2. L- γ -glutamyl-3-carboxy-4-nitroanilide 23 mmol/L, ethylene glycol 900 mmol/L, sodium azide 0.09%.

Warnings and precautions: Warning.

Reagent 1. H319 - H335

P264 – 280 – P305 + P351 + P338 – P337 + P313 – P261 – P271 – P304 + P340 – P312 – P403 + P233 – P405 – P501

Reagent 2. H302 P264 - P270 – P301 + P312 – P330 - P501

Refer to the safety data sheets (SDS) available for download from our website www.linear.es and take the necessary precautions for the use of laboratory reagents.

PREPARATION. The reagents are ready-to-use.

STORAGE AND STABILITY

Store at 2-8°C. Stable until the expiry date stated on the label. After daily use stored tightly closed and protected from light. Do not use reagents over the expiration date. On board the stability is 30 days.

Discard if appear signs of deterioration:

- Presence of particles and turbidity.
- Blank absorbance (A) at 405 nm > 1.400 in 1cm cuvette.

SAMPLE COLLECTION

Serum or EDTA plasma free of hemolysis. Fluoride, citrate and oxalate inhibit γ -GT activity². The enzyme in the sample is stable for at least 1 week at 2-8°C and for at least 2 months when frozen. Only freeze once. Discard contaminated specimens. The samples should be handled as potentially infectious in accordance with Good Manufacturing Practices.

INTERFERENCES⁶

- Lipemia (intralipid >2.5 g/L) may affect the results.
- Bilirubin (> 10 mg/dL) may affect the results.
- Hemoglobin (> 8 g/L) may affect the results.
- Other drugs and substances may interfere⁶.

INSTRUMENTATION AND MATERIALS

- KROMA analyzer.
- Cleaning solutions Ref. KR18100, KR18300, KR18400.
- Multicalibrator CC/H 8x5 mL Ref. CT19750 (Optional).
- General laboratory equipment.

AUTOMATED PROCEDURE

A graphic display pictures the specific sets corresponding to the technical application outlined for this test.

Any new application, to the instrument should be validated to confirm that results meet the analytical performance of the method.

It is recommended to validate periodically the instrument.

CALIBRATION

With factor. The use of theoretical factor is recommended to calculate the activity.

With calibrator. It is optional, it is only necessary if you do not apply the factor. Calibration must be performed with two points (S1: distilled water and S2: Calibrator). Verify the working reagent blank every day before its use.

It is recommended to calibrate the assay at least every month, following preventive maintenance of the analyser, when using a new reagent lot number, or when quality control results are out of range.

CALCULATION

The analyzer automatically calculates the concentration of each sample by theoretical factor or calibrator.

RESULTS

Samples with $\Delta A/\text{min}$ exceeding 0.200 at 405 nm should be diluted 1:10 with saline and assayed again. Multiply the results by 10.

If results are to be expressed as SI units apply: U/L x 16.67 = nkat/L

EXPECTED VALUES⁴

Temperature	37°C	30°C	25°C
Men	10-50 U/L (167-834 nKat/L)	7-35 U/L (117-538 nKat/L)	5-25 U/L (83-417 nKat/L)
Women	8-35 U/L (133-583 nKat/L)	6-25 U/L (100-417 nKat/L)	6-25 U/L (83-300 nKat/L)

It is recommended that each laboratory establishes its own reference range.

QUALITY CONTROL

To ensure adequate quality control (QC), each run should include a set of controls (normal and abnormal) Ref. CT19800-CT19850 with assayed values handled as unknowns. Each laboratory should establish its own quality control scheme and corrective actions if controls do not meet the acceptable tolerances.

DIAGNOSTIC CHARACTERISTICS

Gamma-glutamyl transferase is the most sensitive enzymatic indicator available of hepatobiliary disease. The higher increments are found in cases of intrahepatic or posthepatic biliary obstruction, being more sensitive than alkaline phosphatase when detecting obstructive jaundice, colangitis, and cholecystitis. Elevated levels are noted in the sera of patients with *alcoholic cirrhosis* and from people who are heavy drinkers. The enzyme levels are important in detecting alcohol induced liver disease correlating well with the duration of the drug action.

PERFORMANCE CHARACTERISTICS

Performance characteristics are available on request.

NOTES

1. Clinical diagnosis should not be made on findings of a single test result but should integrate both clinical and laboratory data.
2. The assay procedure must be followed closely, failure to follow the procedure may lead to inaccurate results.
3. To preserve the good functionality of the kit, do not mix different lots neither reagent's remains.
4. Disposal: Used materials should be discarded according to local regulations.

BIBLIOGRAPHY

1. IFCC Primary Reference Procedures for the Measurement of catalytic activity Concentrations of Enzymes at 37°C. part 6. reference procedure for the measurement of catalytic concentration of γ -glutamyltransferase. *J. Clin. Chem. Clin. Biochem./Since* **40**, (2002).
2. Kryszewski, A. J. J., Neale, G., Whitfield, J. B. B. & Moss, D. W. W. Enzyme changes in experimental biliary obstruction. *Clin. Chim. Acta* **47**, 175-182 (1973).
3. Szasz, G., Rosenthal, P. & Fritzsche, W. Die 1-Glutamyl-Transpeptidase-Aktivität im Serum bei hepatobiliären Erkrankungen. *DMW - Dtsch. Medizinische Wochenschrift* **94**, 46-61 (1969).
4. Tietz, N.W. Fundamentals of Clinical Chemistry, p.940. W.B. Saunders Co. Philadelphia, PA. (1987).
5. Friedman y Young. Effects of disease on clinical laboratory tests. 5th ed. AACC (Press 2000).
6. Young DS. Effects of drugs on clinical laboratory tests, 5th ed. AACC Press, 2000.

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REFERENCIAS

1. IFCC Primary Reference Procedures for the Measurement of catalytic activity Concentrations of Enzymes at 37°C. part 6. reference procedure for the measurement of catalytic concentration of γ -glutamyltransferase. *J. Clin. Chem. Clin. Biochem./Since* **40**, (2002).
2. Kryszewski, A. J. J., Neale, G., Whitfield, J. B. B. & Moss, D. W. W. Enzyme changes in experimental biliary obstruction. *Clin. Chim. Acta* **47**, 175-182 (1973).
3. Szasz, G., Rosenthal, P. & Fritzsche, W. Die γ -Glutamyl-Transpeptidase-Aktivität im Serum bei hepatobiliären Erkrankungen. *DMW - Dtsch. Medizinische Wochenschrift* **94**, 46-61 (1969).
4. Tietz, N.W. Fundamentals of Clinical Chemistry, p.940. W.B. Saunders Co. Philadelphia, PA. (1987).
5. Friedman y Young. Effects of disease on clinical laboratory tests. 5th ed. AACC (Press 2000).
6. Young DS. Effects of drugs on clinical laboratory tests, 5th ed. AACC Press, 2000.

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Linear Human Multisera Normal CT19800
Linear Human Multisera Abnormal CT19850

DETERMINACIONES NUMBER OF TESTS

200 tests/kit, Ref. KR10190
1000 tests/kit, Ref. KR10192
(no se considera el volumen muerto).
(dead volume is not taken in consideration).

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Filters		Incubation/Reading time [sec]			Reference range		
F1	405nm	F2	not used			Sample type	Serum
Volumes [microlitres]		Substrate Start			Patient type		
Sample	20	Substrate Start			Female		
R1	160	R1	30 ml	R1, S -> R2	2nd Incub. time	Final incub.	
R2	40	R2	20 ml	32	32	96	
R3	0	R3	20 ml	Kinetic reading time			
					168		
		Kinetic/Fixed Time data			Instrument Factor (Y = aX + b)		
		Substrate depletion			a		
		Fit limit			b		
		1			1.153		
		0.95			-9.930		
					Controls		
					C1 C2 C3		
					No. of SD for QC ref range		
					3		

Reagents			
Blank Abs (min; max)	-999	9999	☒ Include blank in calc.
Reagent linearity	9999		
Detection limit	-999		

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